

# Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days <sup>1</sup>



Control	Lot	
Tri-Level	32312-5K	2024-05-03
Low	32312-51	2024-05-03
Normal	32312-52	2024-05-04
High	32312-53	2024-05-10

## MEDONIC

		CONTROL	L
		LOT	32312-51
Medonic M-series/M32		Mean <sup>2</sup>	± Range <sup>3</sup>
RBC	10 <sup>12</sup> /L	2.22	0.12
MCV	fL	64.3	7.0
RDW	%	14.3	4.5
Hct	%	14.3	3.0
Plt	10 <sup>9</sup> /L	83	15
MPV	fL	11.4	1.5
WBC	10 <sup>9</sup> /L	3.4	0.3
Hgb	g/dL	5.4	0.3
MCH	pg	24.3	2.0
MCHC	g/dL	37.8	4.0
Lymph#	10 <sup>9</sup> /L	1.5	0.4
Mid#	10 <sup>9</sup> /L	0.2	0.3
Gran#	10 <sup>9</sup> /L	1.7	0.8
Lymph	%	45.5	7.0
Mid	%	5.0	5.0
Gran	%	49.5	7.0

		CONTROL	N
		LOT	32312-52
		Mean <sup>2</sup>	± Range <sup>3</sup>
		4.20	0.18
		79.9	7.0
		15.3	4.5
		33.6	3.5
		237	30
		11.4	1.5
		8.6	0.6
		11.9	0.4
		28.3	2.0
		35.4	4.0
		4.1	0.6
		0.4	0.5
		4.1	1.0
		47.7	5.0
		5.1	5.0
		47.2	5.0

		CONTROL	H
		LOT	32312-53
		Mean <sup>2</sup>	± Range <sup>3</sup>
		5.13	0.22
		91.5	7.0
		13.3	4.5
		46.9	4.0
		519	60
		11.8	1.5
		20.9	1.8
		16.2	0.5
		31.6	2.0
		34.5	4.0
		12.8	1.5
		1.0	1.0
		7.1	2.5
		61.2	5.0
		5.0	5.0
		33.8	5.0

- 1
- bg Трайност на отворен флакон 14 дни (bg)
  - hr Stabilnost otvorene bočice 14 dni (hr)
  - cs Stabilita otevrené ampulky 14 dní (cs)
  - da Åben r r stabilitet 14 dage (da)
  - et Stabiilsus avatud vialis 14 p evade (et)
  - fr Stabilit  en flacon ouvert 14 jour (fr)
  - de Stabilit t ge ffneter Flaschen 14 tage (de)
  - el Διατ ρηση δειγματος μετ  την αποσφραγιστ  14 ημερ  (el)
  - it Stabilit  della fiala aperta 14 giorni (it)
  - lv Atv rt flakonu stabilit ti 14 dienas (lv)
  - lt Stabilumas atidarius buteliuk  14 dienas (lt)
  - no  pen r r/glass stabilitet 14 dager (no)
  - pl Trwa o c po otwarciu fiolek 14 Liczba dni (pl)
  - pt Estabilidade ap s abertura do frasco 14 dias (pt)
  - ro Termenul de valabilitate al fiolei desfacute 14 zile (ro)
  - ru Стабильность вскрытого флакона 14 дней (ru)
  - sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
  - sk Stabilita otvorenej injekcnej liekovky 14 dni (sk)
  - sl Stabilita otvorenej injekcnej liekovky 14 dni (sl)
  - es Estabilidad de la c psula abierta 14 d as (es)
  - sv H llbarhet f r  ppnad flaska 14 dagar (sv)
  - tr A ık ŐiŐe stabilitesi 14 g n (tr)

- 2
- Sredno
  - Sredstvo
  - Pr m r
  - Middelv rdi
  - Keskmine
  - Moyenne
  - Mittelwert
  - M Ő 
  - Media
  - Noz m 
  - Vidurkis
  - Middelverdi
  - Őrednia
  - M dia
  - Medie
  - Srednee
  - Srednja vrednost
  - Priemer
  - Povpre no
  - Media
  - Medelv rde
  - Ortalama

- 3
- Ob vat
  - Raspon
  - Rozsah
  - Omr de
  - Vahemik
  - Intervalle
  - Bereich
  - Αναμεν μενο  ρος α
  - Intervallo
  - Diapazons
  - Intervalas
  - Omr de
  - Zakres
  - Intervalo
  - Interval
  - диапазон
  - Opseg
  - Rozsah
  - Obmo je
  - Intervalo
  - Intervall
  - Aral ı

25122-4



<http://www.medonic.se/support/download>



# For Medonic M-series / M32 with barcode reader

## M-Series:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

## M32:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: M-series  
Lot: 32312-51  
Prod.: Con-Diff Low  
ID: 3231251+  
Exp.: 3-May-2024

Instr.: M-series  
Lot: 32312-52  
Prod.: Con-Diff Norm  
ID: 3231252+  
Exp.: 4-May-2024

Instr.: M-series  
Lot: 32312-53  
Prod.: Con-Diff High  
ID: 3231253+  
Exp.: 10-May-2024





